

Calibration Log

HEALTH O METER 600 KL CALIBRATION INSTRUCTIONS

This scale is factory calibrated and does not require calibration prior to use

The calibration procedure may be performed using a known/certified weight in either pounds or kilograms. The default is in pounds, but calibration can be done in kilograms. The following instructions assume the scale is set to pounds. If you prefer to calibrate in kilograms, please call Customer Service at 800-815-6615.

NOTE: Calibration weight must be no less than 60 lb (27kg) and no more than 300 lb (150 kg).

1. Turn the scale on. While “START” is displayed (for 3 seconds); Press **ZERO – UNITS – ZERO**. “CAL” will show on the display.
2. Press **SEND**. “Load/300.0 lb” will show on display.
3. Press **SEND** to calibrate with 300 lb. or load platform with 60 lb. minimum. If calibrating with a lesser weight than 300 lb. press **WEIGHT/HEIGHT/BMI** to change the weight and then the arrows to change the flashing digit. Then press **WEIGHT/HEIGHT/BMI** to move to the next digit, until you reach the required weight (no less than 60 lb and no more than 300 lb). Then press **SEND**. The display will show “CLEAR”
4. Clear the platform and press **SEND**. The display will show Zero Calibration “---“
5. Wait without touching or shaking the scale until the zero calibration process is finished. Display will show Weight Calibration “Put XXX.X”
6. Load the platform with the required weight and press **SEND**. Display will show “- - - -“
7. Wait without touching or shaking the scale until calibration process is finished. “DONE”/“CLEAR” will show on display.
8. Remove weight from the platform. Scale will reboot. “START” will appear on the display followed by the software version and “0.0”

Clinic Site: _____ Equipment ID Tag# _____

Scale Type: Adult Scale

	Test Date	Test Date	Test Date	Test Date
Scales				
Adult				
Sum of weights	Results	Results	Results	Results
Calibration Date				
Calibration Date				
Calibration Date				