

Calibration Log

Seca 374, Calibration Instructions

Calibration for the Seca 374 electronic scale should be performed annually.

Before attempting to calibrate, make sure you have a reproducible reading. For example, apply weight to the platform five times in a row. Record the reading each time. The weight must not vary more than +/- one increment. If it does, the scale may have a mechanical problem in the base instead of a calibration problem.

1. Press and hold the lb/kg button and then press the start button until the scale comes on. A single digit will flash in the display. This indicates the number of times the scale has been calibrated.
2. Release both buttons
3. Press and hold the lb/kg button immediately for 5 seconds, then release.
4. The display will show inc. for increase or dec. for decrease, then 0:0.0
5. Apply 1/2 capacity (25 lbs. Minimum), certified test weight(s) to the platform.
6. Press lb/kg button to increase or decrease weight displayed in increments of 1 oz. or 10 grams.
7. To switch between increase and decrease press and hold lb/kg button until inc. (increase) or dec. (decrease) is displayed.
8. When desired weight is displayed press and hold lb/kg button until display shuts off (approximately 5 seconds).

If you need further assistance, call Seca's Service Department @ 1-800-542-7322.

Clinic Site: _____ Equipment ID Tag# _____

Scale Type: Electronic Pediatric Scale

Scales Pediatric	Results	Results	Results	Results	Results	Results
Calibration Date						
Calibration Date						
Calibration Date						
Calibration Date						
Calibration Date						
Calibration Date						
Calibration Date						